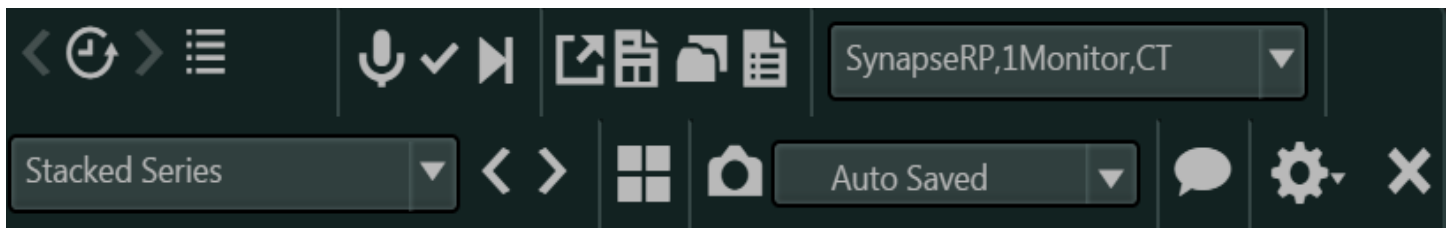

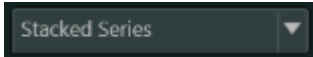



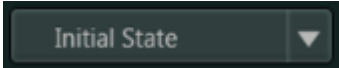





Viewer Toolbar Features



Feature	Icon	Description
Time Warp		This feature addresses the issue of workflow interruption and provides a means by which you can easily return to a study after searching for and opening another study. Use the toolbar buttons for a quick way back to the last study you looked at and easy navigation to every study that you have looked at during your current session.
My History		Click to see a list of your recently viewed studies.
Search		Opens the All Studies Worklist in the Worklist. Use the QBE fields to find another patient.
Send Message to Dictation System		Click to manually notify the dictation system.
Sign Report (F8)		Click to mark the open study dictated and display the next unread study or return to list depending on the Workflow Settings.
Skip Study (F9)		Click to skip the open study without marking dictated and display the next unread study.
External Links Icons		Click to display a list of configured URL icons that supports up to 10 URL icons. See your System Administrator for more information about configuring the URL icons.
Reports		Opens to the Reports pane in PowerJacket.
Open PowerJacket		Opens PowerJacket in a separate window.
DICOM Structured Report and PDF Viewer		Opens the DICOM SR/PDF Viewer.

Reading Protocol Selection Drop-Down Menu		Provides a list of available Reading Protocols for the open study or studies.
Reading Protocol Step Drop-Down Menu		Shows the available steps for the open study or studies.
Previous and Next Step		<p>Click the left button to move to the previous step.</p> <p>Click the right button to move to the next step.</p> <p>Additional step options available on the image context menu: Layout Navigation.</p>
Page Layout		Click to specify the number of viewports displayed on a monitor.
Save Snapshot		<ul style="list-style-type: none"> • Left-click the Snapshot icon to save a new Snapshot.
Snapshots Drop-Down Menu		Indicates that you are using the initial display state of an image. Click to display a list of available Snapshots.
Thinklog™ Chat		Thinklog™ Chat provides the ability for Synapse users to communicate with one another within the Synapse application. Click the Thinklog™ Chat icon from your preferred location in the Worklist, PowerJacket, or Viewer.
Viewer Settings, Users Guide, and About Synapse		<p>Click the arrow to open the drop-down and select</p> <ul style="list-style-type: none"> • Viewer User Settings • Synapse Community • Synapse help (Users Guide) • About Synapse
Close Study		<p>Closes the current study and any prior studies that are open.</p> <p>Note: The Viewer remains open.</p>